

Feasibility and reliability of real-world functional neurophysiology with at-home use of Cumulus' wireless EEG

Cumulus Real-World Neurophysiology Platform

Developed in collaboration with leading pharma companies and KOLs

Cumulus provides full service:


- Protocol/study/SAP design
- On-site training, off-site support
- Full data package
- Reporting and custom analytics

Certified Medical Device

Designed for and with patients and clinicians

Secure automatic upload and QC

Real-time dashboard monitoring of decentralized and home-based data collection.

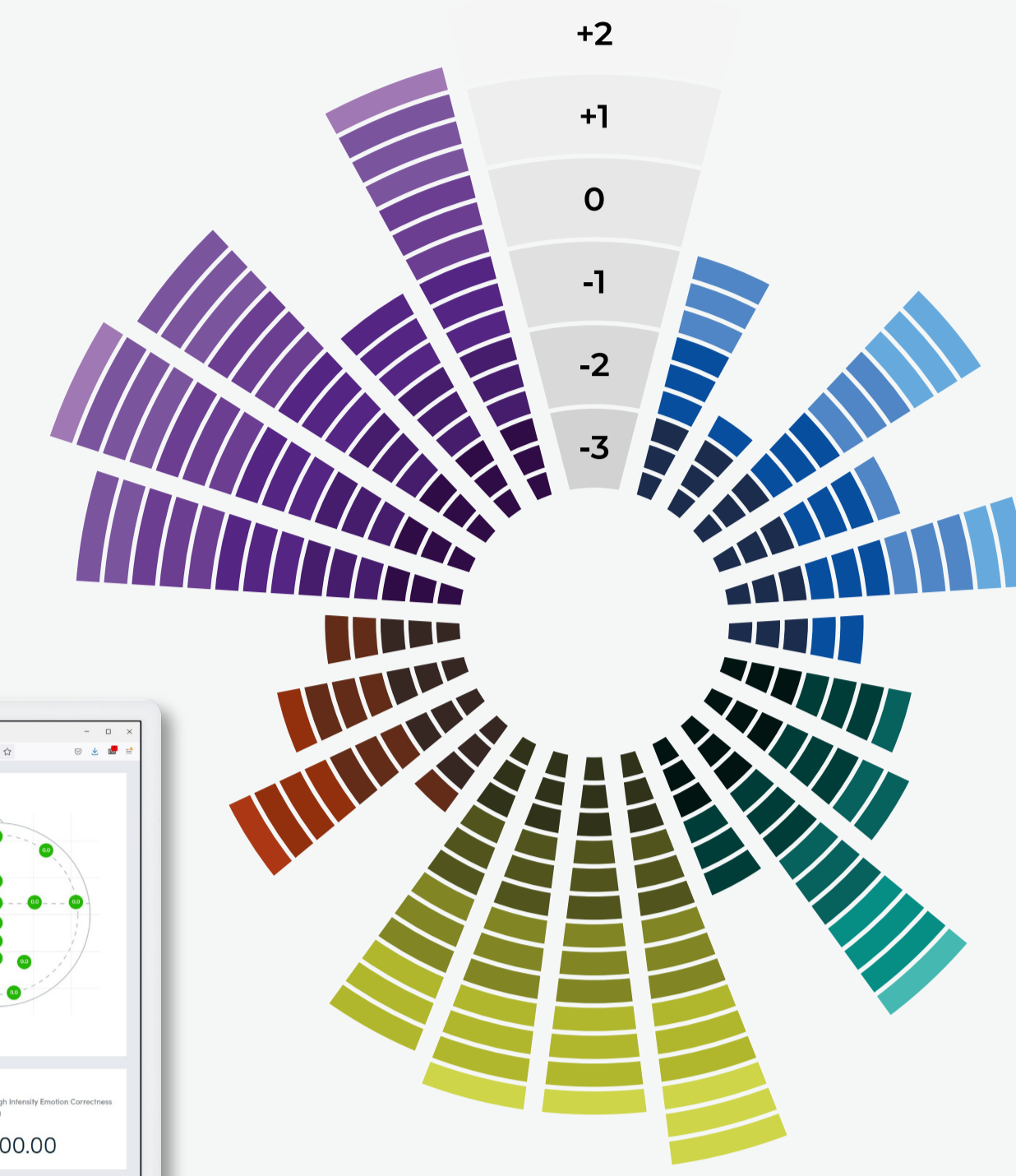

- Cognition**
Episodic memory, executive function
- EEG**
Neuronal integrity, network connectivity
- Sleep**
Sleep quality metrics, sleep staging
- Mood**
Emotional bias, atypical mood
- Language**
Vocabulary, language, organisation, prosody



Introduction

- The recent focus on running decentralised clinical trials has accelerated adoption of methods for "virtual" trials
- Remote survey and digital endpoints may lack objectivity, scientific validation, and grounding in the brain
- Brain imaging and robust cognitive testing still require visits to larger clinics and research centres
- Functional EEG is a technology that can scale to real-world studies, and is clinically informative in neurodegeneration and psychiatry

Research Questions

Q1: Is unsupervised at-home use of functional neurophysiology feasible?

We compare adherence across several real-world studies with controls and patients, and evaluate endpoints for construct validity.

Q2: How do noise levels compare to clinic-based data?

We compare home recorded ERP data to that from a gold-standard benchmark dataset (ERP CORE, Luck Lab, UC Davis)


Q3: How reliable are real-world neurophysiological end-points?

We calculate Intra-Class Correlation, a measure of test-retest reliability, across repeated in-clinic and at-home sessions


Methods & Results

Q1: Is unsupervised at-home use of functional neurophysiology feasible?

Across a number of representative studies, participants were briefly familiarized with the platform, in-person, during enrolment. Control participants, young (av. 26yrs) and old (av. 68yrs), and patients (AD and ALS/FTD), were asked to carry out a series of short (<30m) at-home sessions, repeatedly over a number of months. Adherence data is shown for first 10 at-home sessions. Data for patients are preliminary for studies which are still recruiting. Patients may have assistance of study partner for at-home setup of headset and tablet.

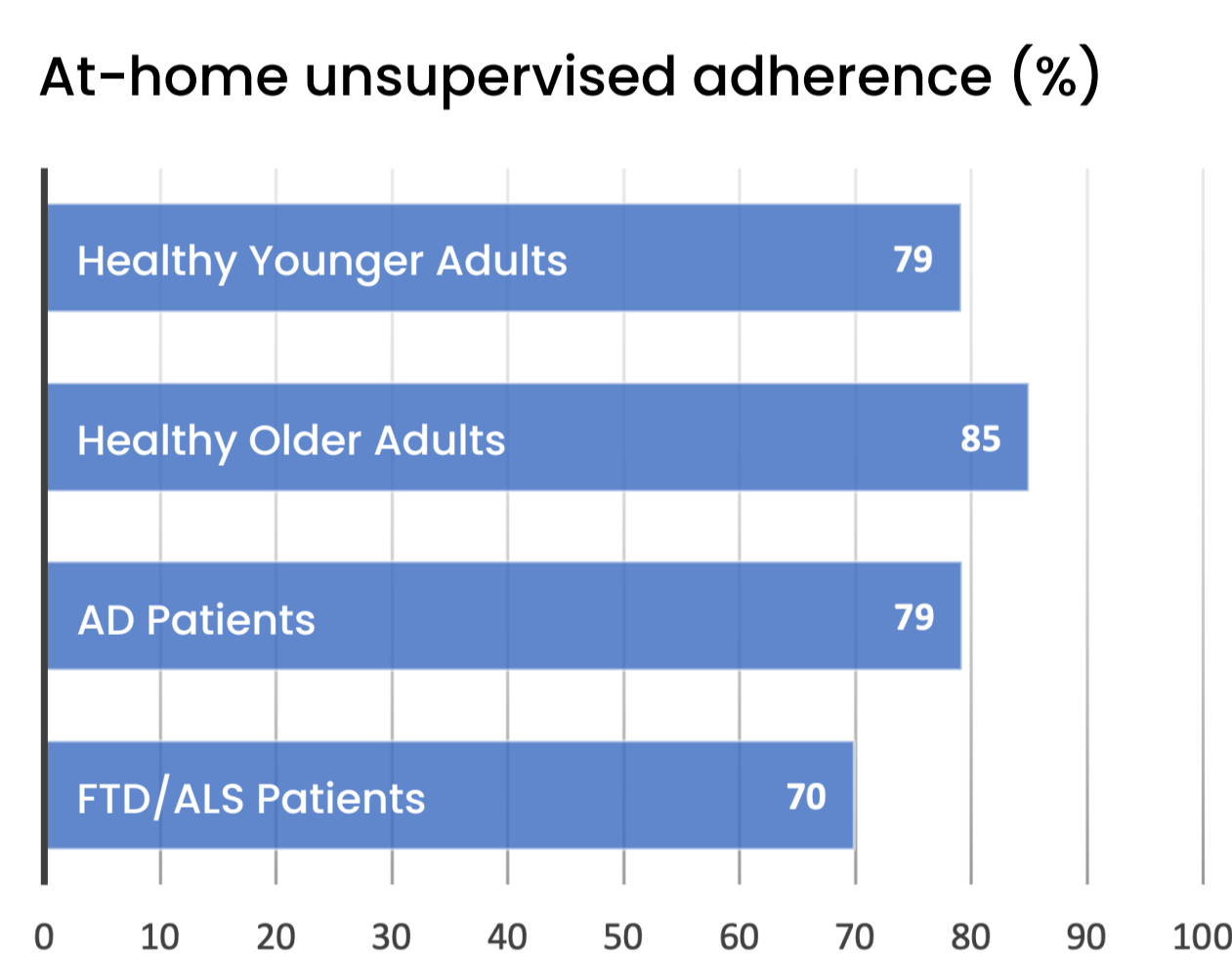

Effect of age (n=89 healthy older adults)

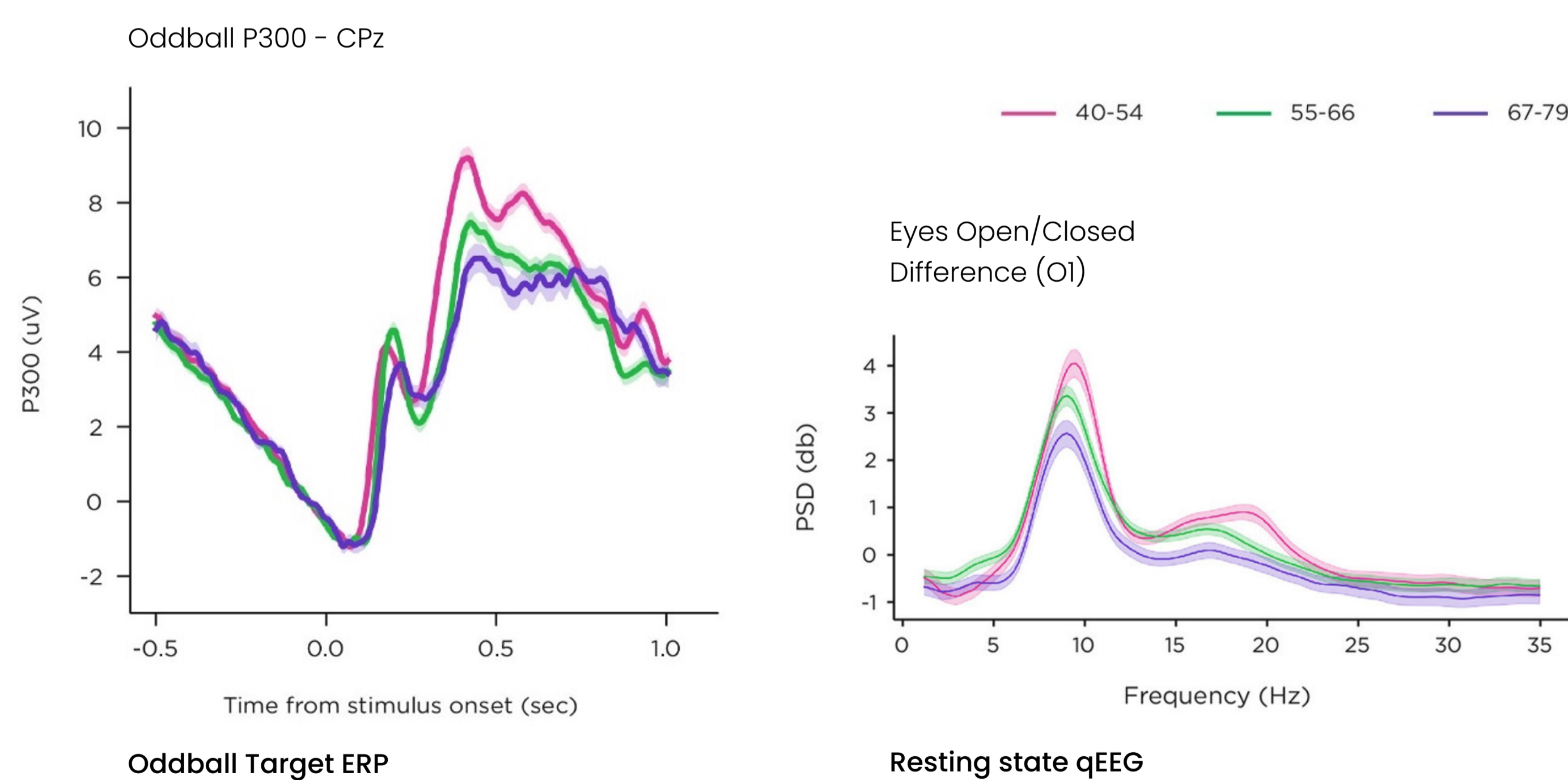


Q2: How do noise levels compare to clinic-based data?


Noise measures from gamified visual oddball and gamified flanker task are compared to a clinic-collected gold-standard benchmark (wet EEG and conventional lab tasks).

N=30 young adult healthy participants

Standardized Measurement Error

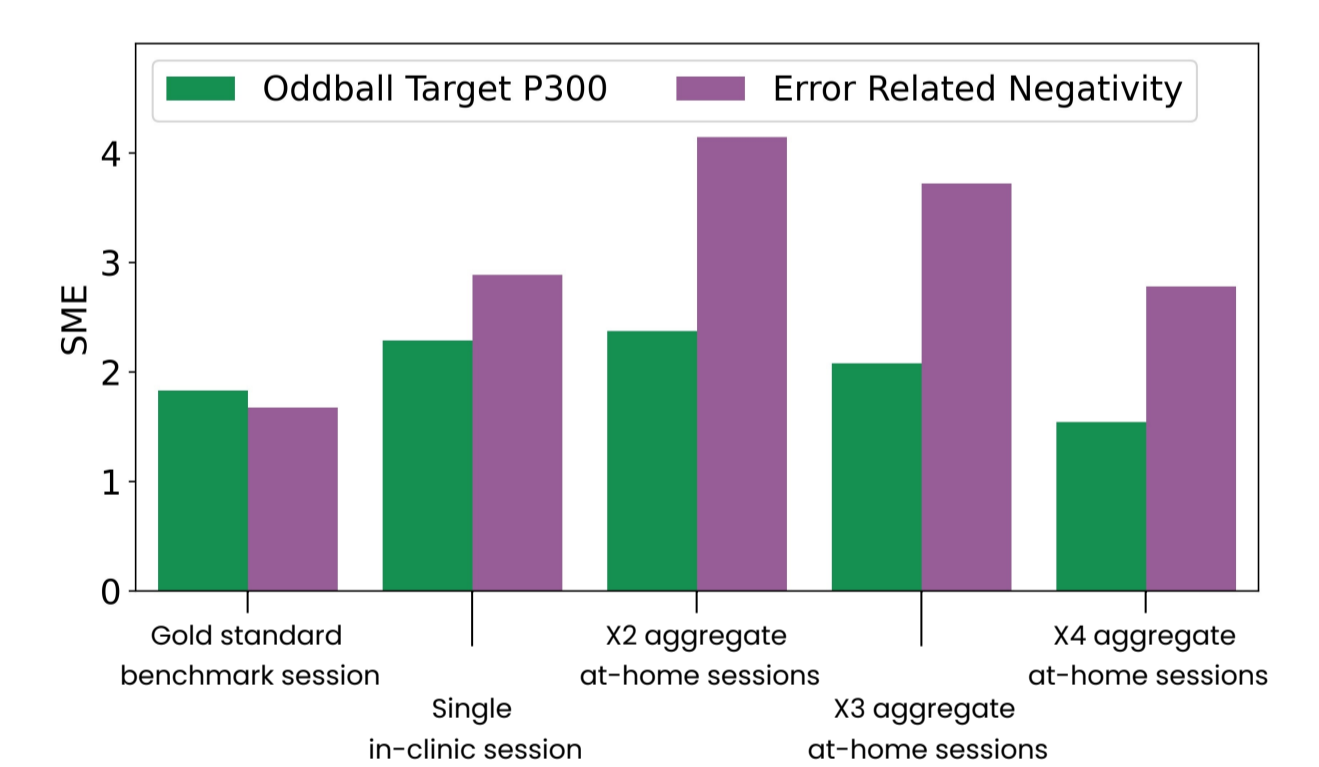

Q3: How reliable are real-world neurophysiological end-points?

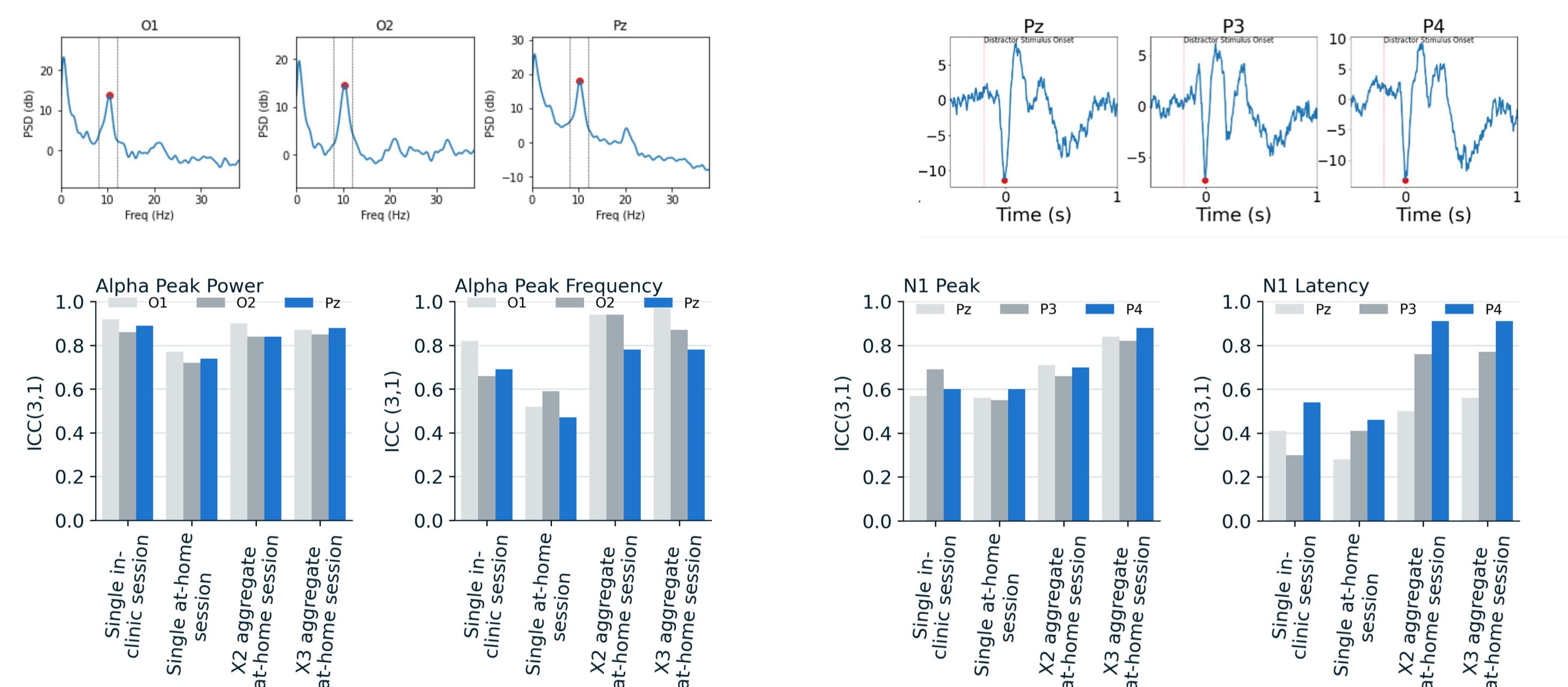

ICC for automatically extracted qEEG power and frequency measures, during a tablet-based resting task paradigm.

ICC for automatically extracted visual ERP amplitude and latency measures, during a gamified flanker task app.

Conclusions

- Older and younger users, including those with a neurological condition, are willing and able to perform short repeated sessions of functional neurophysiology in the home over weeks and months
- Predicted group-level contrasts can be seen at grand-average level, even for subtle changes due to natural aging
- Signal quality and test-retest reliability from a single gamified dry EEG session lags that seen in a conventional clinic setup, but can be compensated for by aggregating over a handful of at-home sessions
- This approach provides a resolution to the dilemma of choice between the quality of clinic-based measures, and quantity of scalable measures
- Pharma precompetitive studies are underway to further validate this approach with longer studies and in relevant patient populations

References

Barbey, F. M., Farina, F. R., Buick, A. R., Danyeli, L., Dyer, J. F., Islam, M. N., ... & Whelan, R. (2022). Neuroscience from the comfort of your home: Repeated, self-administered wireless dry EEG measures brain function with high fidelity. *Frontiers in digital health*, 4.

Cassidy, S. M., Robertson, I. H., & O'Connell, R. G. (2012). Retest reliability of event-related potentials: Evidence from a variety of paradigms. *Psychophysiology*, 49(5), 659-664.

Casson, A. J. (2022). EEG goes home. *Clinical Neurophysiology*.

Horvath, A., Szucs, A., Csukly, G., Sakovics, A., Stefanics, G., Karondi, A. (2018). EEG and ERP biomarkers of Alzheimer's disease: a critical review. *Frontiers in Bioscience (Landmark Ed)*, 23(2), 183-220.

Kappenman, E. S., Farrens, J. L., Zhang, W., Stewart, A. X., & Luck, S. J. (2021). ERP CORE: An open resource for human event-related potential research. *NeuroImage*, 225, 117465.

Luck, S. J., Stewart, A. X., Simmons, A. M., & Rheintulla, M. (2021). Standardized measurement error: A universal metric of data quality for averaged event-related potentials. *Psychophysiology*, 58(6), e13793.

McWilliams, E. C., Barbey, F. M., Dyer, J. F., Islam, M. N., McGuinness, B., Murphy, B., ... & Buick, A. R. (2021). Feasibility of repeated assessment of cognitive function in older adults using a wireless, mobile, dry-EEG headset and tablet-based games. *Frontiers in psychiatry*, 12.

Murphy, B., Barbey, F., Buick, A. R., Dyer, J., Nolan, H. (2020). Demonstration of a novel wireless EEG platform to detect the acute and long-term effects of ketamine, in the lab and in the home. In *Proceedings of FENS International Neuroscience Conference, Virtual/Glasgow*